

Spartan Chemical Company, Inc.

1110 Spartan Drive, Maumee OH 43537 Phone: (800) 537-8990 www.spartanchemical.com

Training Information

Market Segment:

Building Service Contractors, Healthcare, Lodging_Hospitality

HOW TO CLEAN AND DISINFECT WHIRLPOOLS

Check If Applicable: Rx Procedure Cleancheck Procedure	○ None
Tools & Materials Needed:	

- A. Trigger sprayer
- B. White scrub pad
- C. Clean cloths
- D. Gloves and goggles

Products:

- 1. Disinfectant cleaner
- 2. Acid based cleaner

Procedures:

- I. Purpose of Cleaning Whirlpools
 - A. Stop the spread of bacteria, such as those that cause staph and athlete's foot.
- II. Procedure for Cleaning Whirlpools After Every Use
 - A. Put on gloves and recommended personal protection equipment.
 - B. If manufacturer's instructions are not available for disinfecting, a bucket should be used to fill the whirlpool.

This will tell you how many gallons of water it takes to fill the whirlpool. Using a permanent marker,

mark the water line.

- C. Fill the whirlpool to marked line, above pump pickup inlets.
 - 1. Dilute disinfectant in the water according to manufacturer's directions.
 - 2. Turn on circulating pumps to circulate disinfectant.
 - a. Turn off pump.
 - b. Allow disinfectant to stand for 15 minutes.
 - c. Drain.

- d. Refill with water. Turn on the pump for one minute. Shut off the pump and drain.
- 3. Disinfect the rest of the whirlpool using a trigger sprayer filled with a disinfectant cleaner. Spray solution, then agitate with a white scrub pad all the following areas:
 - a. Handrails
 - b. Stairs
 - c. Saddle
 - d. Sidewalls of whirlpool
- 4. Rinse whirlpool with clear water.
- 5. Wipe dry with a clean cloth.
- 6. Now ready to be filled for the next use.
- III. Procedure for Periodic Deep Cleaning of Whirlpool
 - A. Rust and hard water stains can be cleaned using an acid based cleaner.
 - 1. Put on gloves.
 - 2. Drain whirlpool.
 - 3. Spray the acid based product on whirlpool surfaces.
 - 4. Agitate with white scrub pad.
 - 5. Rinse with clear water.
 - 6. Wipe dry with a clean cloth.